


Water Bubbler/Humidifier Combo

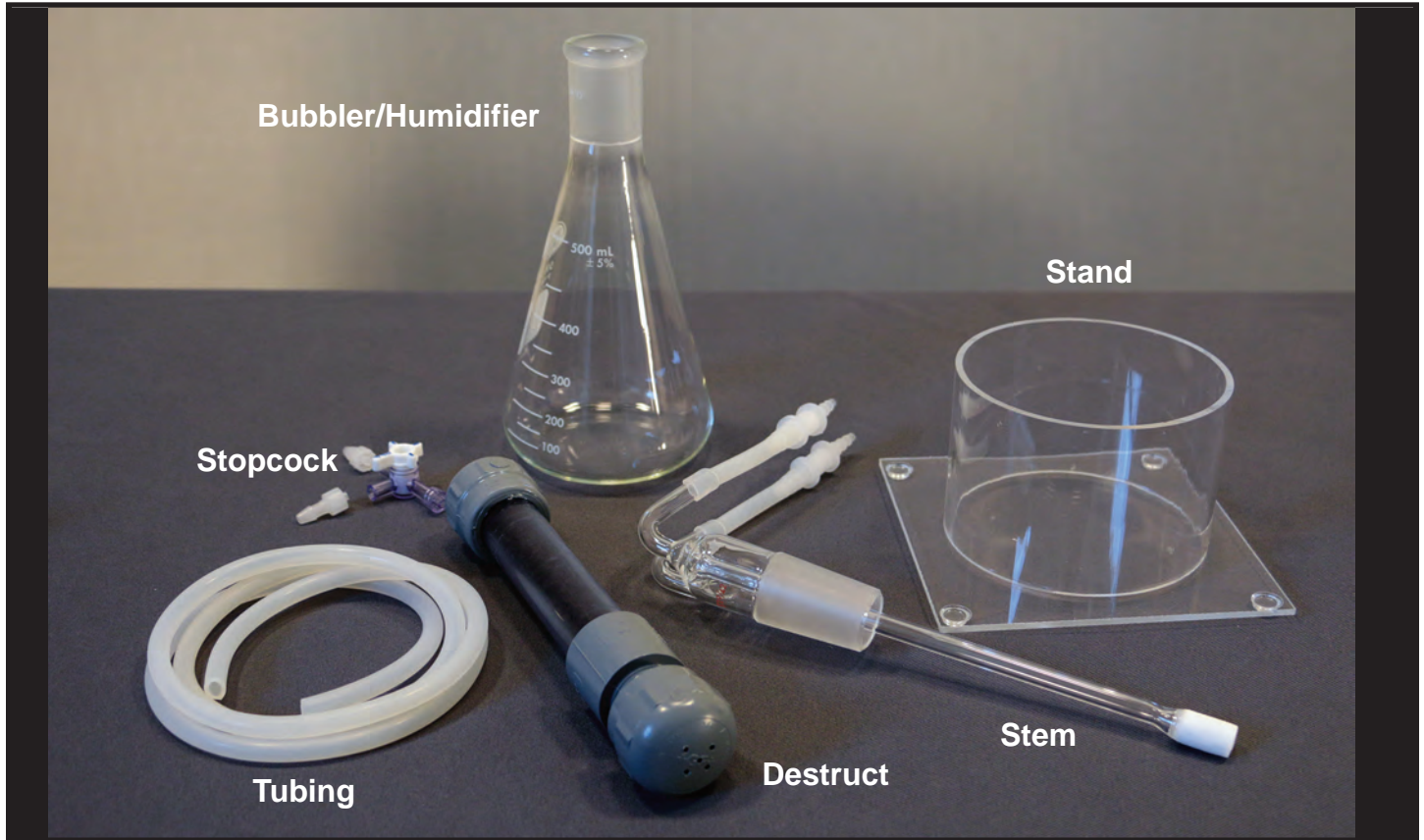

For humidification fill the jar to 200ml. To make ozonated water fill the jar to 500ml. Put the stem back in the jar and put the jar in its stand.

Attach tubing coming from the ozone out of your ozone generator to the left hand straw of the stem (the one that goes into the water). Attach the provided tubing to the right straw of the bubbler and connect the other end to the bottom of the stop cock on the ozone destruct.

Attach tubing from your earscope or other application to the arm of the stop cock on the destruct.

If you are making ozonated water make sure the Off on the stop cock is pointed toward your ozone accessory. If you are using this piece to humidify ozone, make sure the Off on the stop cock is pointed toward the ozone destruct system. Allow water to saturate for several minutes before insufflations. For drinking water, it will take longer to ozonate depending on the amount being used.