

OZONE HAND BAG

If you have large wrists, you might need to use scissors to cut the seal to the proper width. You'll want to make sure you'll still have a good seal once the bag is put on.

Then simply pull the bag in place and make sure the seal is complete and comfortable.

Connect the luer lock fitting to the ozone out port on your ozone generator. Turn your oxygen source to 1/8 LPM and adjust your ozone generator to create 20-40 gamma. Recommended time is 5-10 minutes.

When you're done, turn off the ozone generator and then your oxygen source. You can either remove the bag or wait a few more minute to absorb any ozone left in the bag.



View our video using the QR code or visit
<https://youtu.be/RqTcrQ7L8U>




OZONE HEAD BAG

Note: If using scissors, be careful not to slice the seal itself. If the seal is too tight, you can cut a larger circle. Be advised that cutting the seal in any other way can lead to the formation of larger tears.

Place the bag on your head and press out all the air. You can slide a finger under the seal to let air escape. Connect the tubing to the ozone out port on your ozone generator and run it at your preferred output and time or as directed by your healthcare practitioner.

View our video using the QR code or visit www.promolife.com/headbag

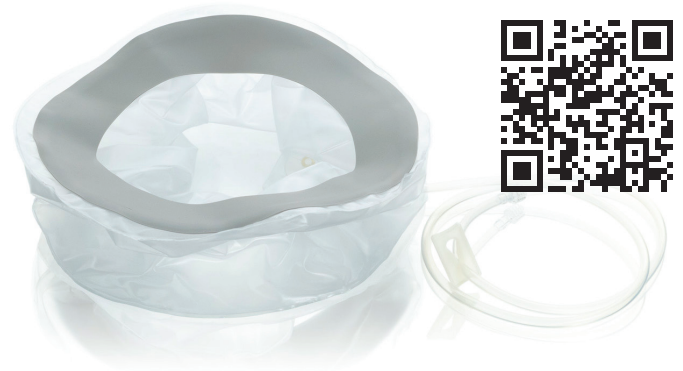

OZONE PENILE BAG

If using the bag around the shaft of the penis, no modification should be needed. Simply pull the bag in place and make sure the seal is complete and comfortable.


To ozonate the entire male genitalia, use scissors to widen the opening. You'll want to make sure you'll still have a good seal once the bag is put on.

Connect the luer lock fitting to the ozone out port on your ozone generator. Turn your oxygen source to 1/8 LPM and adjust your ozone generator to create 20-40 gamma. Recommended time is 5-10 minutes.

When you're done, turn off the ozone generator and then your oxygen source. You can either remove the bag or wait a few more minute to absorb any ozone left in the bag.



View our video using the QR code or visit
<https://youtu.be/e6poHElo-rQ>

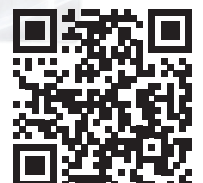


REUSABLE LIMB BAG WITH BUILT-IN SEAL - ARM OR LEG

To use, remove the tab over the opening. If using scissors, be careful not to slice the seal itself.

If the seal is too tight, you can cut a larger circle. Be advised that cutting the seal in any other way can lead to the formation of larger tears.

Place your arm or leg into the bag and adjust as needed. Connect the tubing to the ozone out port on your ozone generator and run it at your preferred output and time or as directed by your healthcare practitioner.

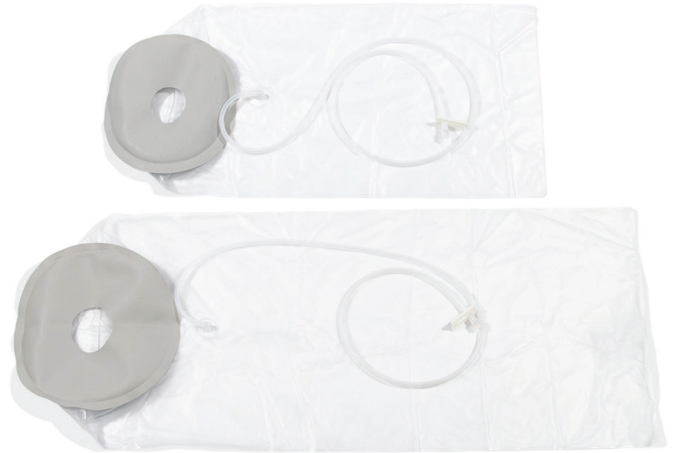

OZONE SHORTS

After you put on the shorts, use the drawstring tabs to tighten or loosen. Once a tight fit is achieved, press out as much air as possible out so the shorts are close to the body.

The pre-installed tube comes with a male Luer lock connector and will attach to a female Luer lock. All ozone out ports on Promolife ozone generators come standard with female Luer lock connectors.

After you connect the Luer lock fitting to the ozone out port on your ozone generator, turn your oxygen source to 1/8 or 1 /4LPM and adjust your ozone generator to create 20-40 gamma. Recommended time is 5 to 10 minutes.

When you're done, turn off the ozone generator and then your oxygen source. You can either remove the bag or wait a few more minutes to absorb any ozone left in the bag.

For usage information please visit the product page. Single user only. If cleaning is required, use a damp cloth.



View our video using the QR code or visit
www.promolife.com/ozoneshorts

